

NINA KUMAR DDS

UltraEZ™ INSTRUCTIONS FOR USE



STEP 1: Express UltraEZ gel into the custom tray. A complete arch should use most of the contents of one syringe. If sensitivity is localized, place gel in affected area(s) and a small amount in the molar region to hold the tray in place.



STEP 2: Make sure teeth are brushed before placing the loaded tray on the arch & gently press the tray edges against the arch labially and palatally.



STEP 3: Remove excess gel with a soft toothbrush or clean finger.



STEP 4: With the tray in place, treatment time begins now. Keep in for 15 min to 1 hour.



STEP 5: At the end of treatment, brush the teeth, being careful not to swallow the UltraEZ gel, and use a soft toothbrush and cold water to clean the tray.